01/2022


12039-20 / 9

Made in the Netherlands

www.push.eu

The Netherlands  
Nea International 31  
Europaplein 61  
6199 AB Maastricht-Airport

2.20.1

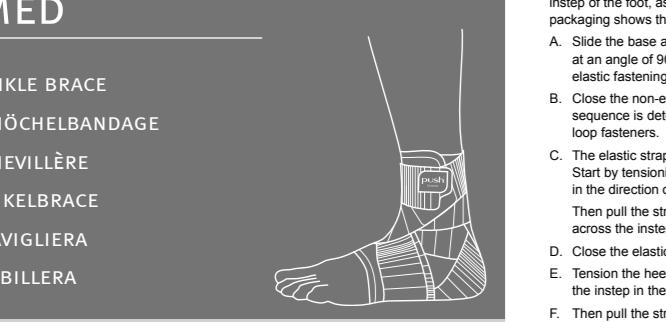## PRODUCT INFORMATION

EN • DE • NL • FR • IT

ES • PT • CS • PL • HU

JA • SV • DA • NO • FI

## ENGLISH

## DEUTSCH

## NEDERLANDS

## FRANÇAIS

## ITALIANO

## ESPAÑOL

## PORTUGUÊS

## WWW.PUSH.EU

## PUSH MED ANKLE BRACE

## INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

Your new ankle brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med ankle brace increases ankle stability and helps control pain. This increases physical strength. Mechanical stabilisation, instellable druck and good position retention constitute the basis of this brace.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfaction used.

Read the product information carefully before you use the Push med ankle brace. If you have any questions, please email us at info@push.eu.

## PRODUKTEIGENSCHAFTEN

Your Push med ankle brace offers a high level of support and protection during the treatment or prevention of ankle injuries. It can be used in a wide range of cases and conditions. The combination of a zipper with a hook-and-loop fastening mechanism ensures that the sock can be easily fitted. Non-elastis straps can be used to set the pronation and supination limits without any impediments for walking. The elastic straps with integrated heel lock, according to the functional tape concept, offer individually adjustable compression around the ankle. The silicone application on the inside of the bracket stimulates the proprioception of the ankle's proprioception. This is done to anticipate an imminent trauma.

The ankle brace has a perfect fit and can be worn in practically any shoe. 'Comfort zones' of padded material are provided around vulnerable areas such as the instep and the heel.

The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the inside feel soft and keeps the skin dry. The Push med ankle brace is easy to apply and can be washed without any problem.

## INDICATIONS

- Moderate to serious instability
- Acute lateral ankle ligament trauma
- Stable ankle fracture
- Follow-up treatment of ankle fractures
- Postsurgical treatment of ligament injuries or fractures
- Capsular irritation
- Sehnenentzündung
- Tendinitis
- Arthrosis mit Instabilität
- Secondary prevention of ankle ligament trauma
- Impingement (capsular clamping; anterior and posterior)
- Football ankle

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

## DETERMINING SIZE / FITTING

The ankle brace is available in 5 sizes. Position the foot at an angle of 90° in relation to the lower leg. Then measure the circumference around the heel and the instep of the foot, as shown in the illustration on the packaging. The table on the packaging shows the correct brace size for your foot.

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A gentle spin-dry cycle can be used; after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

## WICHTIG

- Für eine richtige Anwendung der Push Bandage kann eine Diagnose eines Arztes oder Therapeuten notwendig sein.
- Push braces were designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Try on the brace before purchasing to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to consult for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Check your Push brace before each use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the appropriate regulatory body in the user's country of residence.

Push® is a registered trademark of Nea International bv

For details of the distributor in your country, refer to [www.push.eu/distributors](http://www.push.eu/distributors)

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